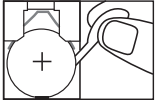
- | | | | |
|------------|---|------------|--|
| D | Glaswaage
Gebrauchsanweisung | GR | Γυάλινη ζυγαρία
Οδηγίες χρήσης |
| GB | Glass scale
Instruction for Use | DK | Glasvægt
Betjeningsvejledning |
| F | Pèse-personne en verre
Mode d'emploi | S | Glasvåg
Bruksanvisning |
| E | Báscula de vidrio para personas
Instrucciones para el uso | N | Glassvekt
Bruksveiledning |
| I | Bilancia in vetro
Istruzioni per l'uso | FIN | Lasivaaka
Käyttöohje |
| TR | Cam Terazı
Kullanma Talimatı | CZ | Skleněná váha
Návod k použití |
| RUS | Стеклянные весы
Инструкция по применению | SLO | Steklena tehtnica
Navodila za uporabo |
| PL | Waga szklana
Instrukcja obsługi | H | Üvegmérleg
Használati utasítás |
| NL | Weegschaal met glazen weegplateau
Gebruikershandleiding | RO | Cântar de sticlă
Instrucțiuni de utilizare |
| P | Balança de vidro
Instruções de utilização | | الميزان الزجاجي
دليل الاستعمال |

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- (GB)** Getting started
- (F)** Mise en service
- (E)** Puesta en marcha
- (I)** Messa in funzione
- (NL)** Ingebruikname

- (P)** Colocação em funcionamento
- (TR)** İlk çalıştırma
- (GR)** Έναρξη λειτουργίας
- (DK)** Ibrugtagning

- (S)** Börja använda vågen
- (N)** Bruk
- (FIN)** Käyttöönotto
- (RUS)** Ввод в эксплуатацию
- (PL)** Uruchomienie

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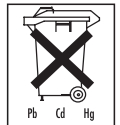
(GB) General advice

- Max. capacity 180 kg (330 lb / 24 st), Graduation 100 g (0.2 lb / 0.01 st).
- Cleaning: You can clean the scale with a damp cloth and a little washing up liquid, if required. Never immerse the scale in water or rinse it under running water.
- Protect the scale against knocks, damp, dust, chemicals, drastic changes in temperature, electromagnetic fields and sources of heat that are too near.
- Never step onto the outer edge of the scale with one foot as this could cause the scales to tip!
- Keep packaging material away from children!
- Not intended for commercial use.
- Should you have any questions regarding operation and usage of our appliances, please contact your local retailers or Customer Service.
- Before making a claim, please check the batteries and replace them if necessary.
- Repairs must only be carried out by Customer Services or authorised suppliers.
- Spent batteries and rechargeable batteries do not constitute normal household waste! They are considered to be toxic waste and, as such, should be disposed of in special containers, toxic waste collection points or brought to electrical goods dealers. You are legally obliged to dispose of the batteries.
- Note: Batteries containing pollutant substances are marked as follows:
 Pb= Battery contains lead,
 Cd = Battery contains cadmium,
 Hg = Battery contains mercury.

- Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please contact the appropriate local authorities.

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