- (D)** **Glaswaage**
Gebrauchsanweisung
- (GB)** **Glass scale**
Instruction for Use
- (F)** **Pèse-personne en verre**
Mode d'emploi
- (E)** **Báscula de vidrio para personas**
Instrucciones para el uso
- (I)** **Bilancia in vetro**
Istruzioni per l'uso
- (TR)** **Cam Terazı**
Kullanma Talimatı
- (RUS)** **Стеклянные весы**
Инструкция по применению
- (PL)** **Waga szklana**
Instrukcja obsługi
- (NL)** **Weegschaal met glazen weegplateau**
Gebruikershandleiding
- (P)** **Balança de vidro**
Instrukcja obsługi
- (GR)** **Γυάλινη ζυγαρία**
Οδηγίες χρήσης
- (DK)** **Glasvægt**
Betjeningsvejledning
- (S)** **Glasvåg**
Bruksanvisning
- (N)** **Glassvekt**
Bruksveiledning
- (FIN)** **Lasivaaka**
Käyttöohje
- (CZ)** **Skleněná váha**
Návod k použití
- (SLO)** **Steklena tehtnica**
Navodila za uporabo
- (H)** **Üvegmérleg**
Használati utasítás
- (RO)** **Cântar de sticlă**
Instrucțiunile de utilizare
- الميزان الزجاجي
دليل الاستعمال

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- (GB)** Getting started
- (F)** Mise en service
- (E)** Puesta en marcha
- (I)** Messa in funzione
- (TR)** İlk çalıştırma

- (RU)** Ввод в эксплуатацию
- (PL)** Uruchomienie
- (NL)** Ingebruikname
- (P)** Colocação em funcionamento

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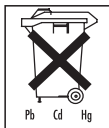
GB General advice

- Max. capacity 180 kg (396 lb / 28 st), Graduation 100 g (0.2 lb / 0.1 st).
 - Cleaning: You can clean the scale with a damp cloth and a little washing up liquid, if required. Never immerse the scale in water or rinse it under running water.
 - Protect the scale against knocks, damp, dust, chemicals, drastic changes in temperature, electromagnetic fields and sources of heat that are too near.
 - Never step onto the outer edge of the scale with one foot as this could cause the scales to tip!
 - Keep packaging material away from children!
 - Not intended for commercial use.
 - Should you have any questions regarding operation and usage of our appliances, please contact your local retailers or Customer Service.
 - Before making a claim, please check the batteries and replace them if necessary.
 - Repairs must only be carried out by Customer Services or authorised suppliers.
 - Spent batteries and rechargeable batteries do not constitute normal household waste! They are considered to be toxic waste and, as such, should be disposed of in special containers, toxic waste collection points or brought to electrical goods dealers.
- You are legally obliged to dispose of the batteries.

- Note: Batteries containing pollutant substances are marked as follows:
Pb= Battery contains lead,
Cd = Battery contains cadmium,
Hg = Battery contains mercury.

- Follow local regulations regarding the disposal of materials. Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any queries, please contact the appropriate local authorities.

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