beurer

MG 800

Fascia-
Releazer®

D	Faszien-Massagegerät Gebrauchsanweisung	2
GB	Fasciae massager Instructions for use	20
F	Appareil de massage des fascias Mode d'emploi	37
E	Aparato de masaje de fascias Instrucciones de uso.....	55
I	Apparecchio per il massaggio fasciale Istruzioni per l'uso	72
TR	Bağdokü masaj cihazı Kullanım kılavuzu.....	89
RUS	Прибор для массажа фасций Инструкция по применению.....	106
PL	Aparat do masażu powięzi Instrukcja obsługi.....	126

CE

ENGLISH

Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Dear customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, gentle therapy, blood pressure/diagnosis, weight, massage, beauty, air and baby care.

With kind regards,
Your Beurer team.

Table of contents

1. Getting to know your device	22
2. Signs and symbols	24
3. Intended use	24
4. Notes	26
5. Device description	28
6. Initial use	29
7. Use.....	30
7.1 Preparing for the application	30
7.2 Switching on the Fascia-ReleaZer® / selecting the vi- bration setting.....	32
7.3 Exercises	34
8. Maintenance and cleaning	35
9. Disposal	36
10. Technical data	36

Included in delivery

- 1x Fascia-ReleaZer® / 1x micro USB charging cable / 1x mains adapter / 1x training DVD with training exercises / 1x exercise overview

1. Getting to know your device

What are fasciae?

Fasciae are fine, tough connective tissue fibres that pass through our entire body like a three-dimensional network. Human fasciae surround all muscles, bones, organs and nerve cells in the body and ensure that when we move, the individual parts of our body glide past one another and remain in place. Fasciae also act as links between the muscle fibres and ensure that we are able to use our muscle strength. The strain energy between the tendons and bones is stored and then suddenly released upon relaxing.

Fasciae shorten and harden if they are not used on a regular basis (similarly to muscles). The structures of the fasciae stick together and thicken, which causes them to become porous and lose their elasticity. This results in small tears and inflammation, which also affects the transmission of force and your coordination. In isolated cases, thickened fasciae may also press down on the nerves and cause discomfort. There may be a variety of reasons as to why fasciae have shortened or stuck together, such as a lack of movement, protective postures or psychological stress.

How does the Fascia-ReleaZer® work?

If fascia tissue has become stuck together or hardened, you can improve its suppleness and elasticity by alternately applying pressure and tension. By using the Fascia-ReleaZer® as a self-help device, you can positively stimulate

these receptors in our tissue. This will increase and speed up the regenerative capacity of the tissue.

Your blood supply will also be improved and your nerves will function better. The stimulation from the Fascia-ReleaZer® allows new nutrients to reach the cells and makes it easier to transport away waste. In this way, the tissue is replaced and the elasticity and resilience of the tissue in the treated area are improved.

According to study:

Randomised, Double Controlled, Standardised, Clinical Study, Europe.

Performed by:

Center for Integrative Therapy, Germany.


Institute of Medical Psychology and Behavioral Neurobiology, University of Tübingen, Germany. Research Institute on Health Sciences (IUNICS), University of Balearic Islands, Palma, Spain.

Another way to boost the elasticity of the tissue is with the vibration massage, which relaxes the muscles. This reduces stiffness, which in turn improves your movement.

The massage treatment must feel pleasant at all times. A slight haematoma may well occur – this is a positive therapeutic effect as the use of the Fascia-ReleaZer® can selectively encourage your blood flow. In isolated cases areas of redness may also occur on the skin. If you experience excessive skin irritation, stop using the Fascia-ReleaZer® and consult a doctor.

To ensure that the treatment is pleasant and tailored to you, we recommend the use of the sensation/pain scale. This will help you to find the ideal level of pressure for you. Since each person has a different perception of pain, we recommend a massage treatment in the recommended range (2-8). If the sensation rating lies between 0 and 2, the treatment will not be particularly effective as the

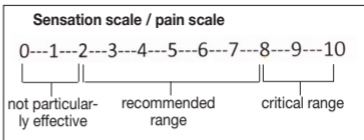Fascia-ReleaZer® is unable to penetrate deep into the tissue. We would advise against a massage treatment in the critical range (more than 8), since excessive pressure is exerted on the tissue in this range.

2. Signs and symbols

The following symbols appear in these instructions for use:

	WARNING Warning instruction indicating a risk of injury or damage to health
	IMPORTANT Safety note indicating possible damage to the device/accessory
	Note Note on important information

3. Intended use

WARNING

Only use the Fascia-ReleaZer® for massaging human muscle fasciae. The Fascia-ReleaZer® is intended for private use only. The Fascia-ReleaZer® is not a medical device, rather it is a massage treatment device. The Fas-

cia-ReleaZer® may only be used for the purpose for which it is designed and in the manner specified in the instructions for use. Improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or careless use.

Use of the Fascia-ReleaZer® is prohibited in the following cases:

- Not with changes caused by disease or injury in the back region (e.g. disc problems, open wounds),
- not during pregnancy,
- never while you sleep,
- not in the car,
- not on animals,
- not during activities during which an unexpected reaction may be dangerous,
- not while ingesting substances that reduce reaction times (e.g. painkiller medications, alcohol),
- never longer than 15 minutes (risk of overheating) and allow it to cool off for at least 15 minutes before using it again.

Please consult your doctor if you are unsure whether or not the Fascia-ReleaZer® is suitable for you.

You should consult your doctor before using the Fascia-ReleaZer® if:

- you suffer from a serious illness or are recovering from an operation,
- you suffer from any of the following: common cold accompanied by a fever, varicose veins, thrombosis, phlebitis, jaundice, diabetes, nervous system diseases (e.g. sciatica) or acute inflammations.
- you are fitted with a cardiac pacemaker, implant or other aid,
- in case of pains of unclarified origin.

4. Notes

WARNING


Safety note regarding use of the Fascia-ReleaZer®

- Keep the packaging material away from children. Choking hazard!
- The Fascia-ReleaZer® is not a medical device. It is a massage treatment device.
- The device is intended for private use only.
- Children must not play with the device.
- Before use, ensure that the device and accessories show no visible damage. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.
- This device may be used by children over the age of eight and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely, and are fully aware of the consequent risks of use.
- Repairs must only be carried out by Customer Services or authorised suppliers. Under no circumstances should you open or repair the Fascia-ReleaZer® yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to comply will void the warranty. Never attempt to repair the device yourself!
- Do NOT use the Fascia-ReleaZer® on your head.
- Do NOT use the Fascia-ReleaZer® on your face.
- Do NOT use the Fascia-ReleaZer® on your genitals.
- Only use the device with the supplied accessories.

Notes on handling rechargeable batteries

- If your skin or eyes come into contact with fluid from a rechargeable battery cell, flush out the affected areas with water and seek medical assistance.
- Protect batteries from excessive heat.
- Risk of explosion! Never throw batteries into a fire.
- Do not disassemble, split or crush the rechargeable batteries.
- Only use chargers specified in the instructions for use.
- Batteries must be charged correctly prior to use. The instructions from the manufacturer and the specifications in these instructions for use regarding correct charging must be observed at all times.

5. Device description

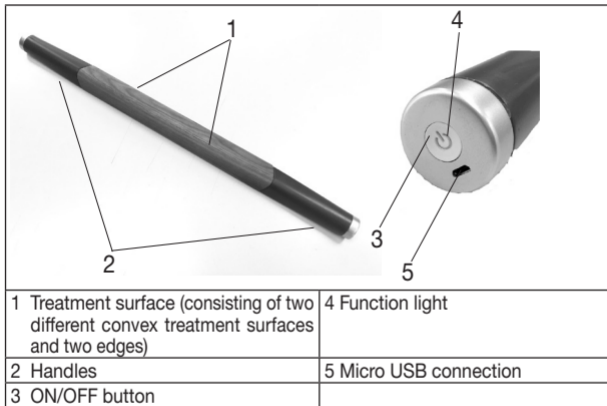Function light status 

Function light shows steady green

= Static vibration

Function light flashes green

= Wave-action vibration

Function light status 	Charging function (charging cable plugged in)
Function light flashes yellow	= Charging
Function light shows steady yellow	= Charging complete
Function light flashes red	= Low battery

6. Initial use

Note

Before using the Fascia-ReleaZer[®] for the first time, let it charge for a minimum of 2.5 hours. Proceed as follows:

- Connect the charging cable to the mains adapter and the Fascia-ReleaZer[®] (*Figure 1*). Then insert the mains adapter into a suitable socket. Alternatively, you can also charge the Fascia-ReleaZer[®] with the charging cable via the USB port on your computer or laptop (*Figure 2*).

Whilst charging, the function light flashes yellow. When the Fascia-ReleaZer[®] is fully charged, the function light shows steady yellow. The battery will last for approx. 11 hours of operation when fully charged. Charge the Fascia-ReleaZer[®] again as soon as the function light flashes red during operation.

Figure 1


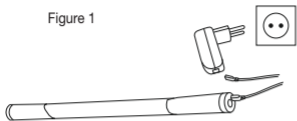
Figure 2

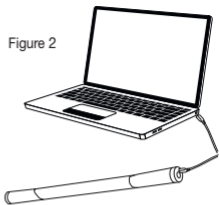7. Use

7.1 Preparing for the application

We recommend treatment with or without plant-based massage oil.

If you have thick body hair, we recommend treatment with plant-based massage oil. However, we generally advise using the device on smooth, hair-free skin.

Do not use any oils or creams with a strong cooling or heating effect, as they may cause skin irritation.

Treatment without massage oil: (recommended if you have no or little body hair)

A massage treatment without massage oil provides a surface massage. This massage treatment activates the reflex zones and treats the surface of the tissue.

Treatment with massage oil: (recommended if you have thick body hair or if you would like a deep massage)

If you opt for massage oils, the massage will penetrate deep into the tissue. This will strongly promote your circulation.

IMPORTANT

Only use plant-based oils as other oils may stain or damage the walnut part of the Fascia-ReleaZer®.

1. Put a couple of drops of plant-based oil in the palm of your hand.
2. Rub the plant-based oil evenly over the area you wish to massage. Before starting the massage treatment, wipe your hands with a cloth to prevent them from slipping during the treatment and to prevent damage to the silicone surface on the handles.
3. After the massage treatment, wipe the wood surface on the Fascia-ReleaZer® and the treated area with a cloth.

The amount of oil to be used depends on the effect you wish to achieve. More oil causes a faster movement and therefore a more intensive reflex action and circulation of blood to the skin.

Less oil causes a slower treatment that penetrates deeper, with a very relaxing effect that loosens the tissue.

It is also possible to combine the two. To do so, apply a larger amount of oil onto the area to be treated and move the Fascia-ReleaZer® over the area quickly. Then wipe off some of the oil with a cloth and treat the same area again, but this time with slow movements.

7.2 Switching on the Fascia-ReleaZer® / selecting the vibration setting

Press the ON/OFF button to switch on the Fascia-ReleaZer®. The function light shows steady green and the first vibration setting is activated.

Action		Vibration setting
1st press of the button	=	The Fascia-ReleaZer® switches on and the function light shows steady green. Static vibration
2nd press of the button	=	The function light flashes green. Wave-action vibration
3rd press of the button	=	The Fascia-ReleaZer® switches off and the function light goes out.

The Fascia-ReleaZer® can be used statically (targeted application) or dynamically (moving application).

Static application:

This method loosens specific areas of hardness. Press down evenly on the same spot approx. 3 times in succession.

Dynamic application:

The movement smooths out the fasciae. Moving the device quickly forwards and backwards provides a surface massage for the upper tissue. Moving the device more slowly allows the massage effect to penetrate deeper. Avoid striking the device against the treatment area as this may result in haematoma.

Treatment surfaces:

We recommend starting the massage treatment with one of the two different convex treatment surfaces. After approx. one week, the massage treatment can be continued with both edges. The edges penetrate deep into the tissue and provide a deep massage treatment.

7.3 Exercises

Note

We recommend treating each area (lower body, mid-body and upper body) three times a week for 15 minutes each time.

When carrying out a session, we recommend training all three areas. However, you may also choose to focus on one of the above areas.

After each massage treatment, wipe down the Fascia-ReleaZer® with a dry cloth or hand towel. Ensure that no traces of oil that could attack the surface are left on the device.

All exercises can be found on our enclosed training DVD and in our exercise overview.

Exercises for the lower body

Lower leg, thighs, shin, calves

Exercises for the mid-body

Lower back (not on the spine), bottom

Exercises for the upper body

Neck, upper and lower arms

8. Maintenance and cleaning

The service life of the Fascia-ReleaZer® depends on careful use.

IMPORTANT

- Protect the Fascia-ReleaZer® from knocks, dust, chemicals, drastic changes in temperature, electromagnetic fields and nearby sources of heat (ovens, heaters).
- Clean the Fascia-ReleaZer® with a dry cloth each time after you have finished using it. Do not use any abrasive cleaning products.
- Cleaning must not be performed by children unless supervised.

9. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life.

Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.


Rechargeable battery disposal

The empty, completely flat rechargeable batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the rechargeable batteries.

The codes below are printed on rechargeable batteries containing harmful substances:

- PB = Battery contains lead,
- Cd = Battery contains cadmium,
- Hg = Battery contains mercury.

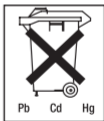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Beurer GmbH • Söflinger Str. 218 • 89077 Ulm, Germany