Beurer GmbH • Söflinger Straße 218 • 89077 Ulm, Germany
Tel.: +49(0)731 / 39 89 - 144 • Fax: +49(0)731 / 39 89 - 255
www.beurer.com • E-Mail: kd@beurer.de

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<http://www.beurer.com/web/de/service/faq/faq.php>

 **Congratulations on the purchase of your new AS 81 Beurer activity sensor!**

D
GB
F
E

You can use the activity sensor with our Beurer BodyShape app if you want to get your body into shape, eat healthily and reach your ideal weight, or with our Beurer HealthManager app if you want to make use of our complete health monitoring system.

Activate the activity sensor before initial use by holding down the button. Synchronise the activity sensor with your smartphone before first use to transfer the basic settings (e.g. time) to your activity sensor.

The activity sensor continually records your physical activity.

All your physical movements are then converted into steps.

The information on the distance covered can therefore differ from the actual distance.

The activity sensor is splash-proof.

Do NOT use the activity sensor in the shower or whilst swimming!

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- GB** Start the app and follow the instructions.

HealthManager: You can either register for free and enjoy the benefits of the Beurer web version (cloud) or continue without registering.

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- GB To synchronise data with your smartphone, press and hold the button for 5 seconds.

Get active!

To guarantee successful synchronisation, ensure that Bluetooth® is activated on your smartphone.

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Important notes

Protect the device from damp. Do not submerge the device in water. Avoid contact with suncream and similar substances as these can damage the plastic parts. Opening the screws or improper handling will void the warranty.

Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

We hereby guarantee that this product complies with the European R&TTE Directive 1999/5/EC. Please contact the specified service address to obtain further information, such as the CE Declaration of Conformity.

The word mark *Bluetooth®* and accompanying logo are registered trademarks of *Bluetooth® SIG, Inc.* Any and all use of these marks by Beurer GmbH is done so under licence. Other trademarks and trade names are the property of the relevant holder.

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